**DENTISCARE LTDA** - Av. Edgar Nelson Meister, 474 - Distrito Industrial - 89219-501 - Joinville - SC - Brasil / Brazil. CNPJ: / Tax ID: 05.106.945.0001-06 INDUSTRIA BRASILEIRA / BRAZILIAN INDUSTRY - Responsável Técnico / Technical Authority: Friedrich Georg Mittelstädt CRC: 13100147 - SC - ANVISA nº 80172310008. / Fabricado no Brasil/Made in Brazil - Marca / Brand: FGM®

☑️ **NUNO FLORES** - Al. Bonifácio Lazzaro Lozano, 3 – PésO 0 – C 2780-125 Oeiras / Portugal - Tel. (351) 21 4439292 - ec.rep@fgm.ltda

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**EN Instruccions for use**

**Composite for anterior and posterior teeth**  
**Professional use only**  
 Read all the information in this manual carefully before using the product. Keep the manual for later reference at least until the product is finished or until there is no more interaction between the product and its last patient.

**Product description**  
 Opallis is a radiopaque, light-curing composite resin with fluorescent and opalescent properties, high positioning power and several options of shades and translucence. It is an aesthetic restorative material used for restorations of anterior and posterior teeth.  
 Opallis was carefully developed to reproduce the polychromatism of teeth in great detail, with great mechanical properties.

**Presentation**  
**Refill:** 1 syringe with 1 resin shade.  
**Basic kit:** 6 syringes  
**Shades:** EA2, EA3, EA3.5, DA2, DA3 and T-Neutral  
**Kit Intero:** 4 syringes, 1 Condac 37% and 1 Ambar.  
**Cor:** EA1, EA2, EA3 and DA2.  
**Clinical kit:** 13 syringes  
**Shades:** EA1, EA2, EA3, EB2, DA1, DA2, DA3, DB2, D-Bleach, T-Blue, T-Neutral, E-Bleach H and Opaque Pearl.  
**Caps:** package containing 20 caps with 0,2g each

**Basic composition**  
 Opallis is a composite resin composed by a monomeric matrix containing Bis (GMA), Bis (EMA), UDMA and TEGDMA. The fillers are a careful combination of Barium-Alumina, silanized silicates and particles of silicon dioxide, camphorquinone as photoinitiator, accelerators, stabilizers and pigments.  
 The composite has particles in the range 40nm to 3,0 microns with an average particle size of 0,5 microns, the total filler content of 78.5% to 79.8% in weight and 57% to 58% in volume of inorganic fillers.  
 The distribution of the sizes of the particles allows an adequate filling of the resin which contributes for its high mechanical resistance as well as resistance to wear. Such characteristics are necessary for restorations of posterior teeth and for easy polishing, generating a

restoration with flat surface and high brightness for anterior teeth.  
**Product indication**  
 The composite is suitable for use in permanent and deciduous teeth:  
 - Direct restorations in anterior and posterior teeth Classes I, II, III, IV, V and VI;  
 - Direct veneers with composites;  
 - Cementation of tooth fragments;  
 - Core build-ups;  
 - Teeth splinting;  
 - Diastema closing or reduction;  
 - Modification of teeth 's shape (e.g.: conoid teeth);  
 - Porcelain/composite repairs;

**Opallis shade system**  
 Opallis uses the VITA® Classical and/or Opallis shade guide for its shades of enamel and dentin. Besides, it also presents composites with shades of elevated translucence and opacity, and shades for bleached teeth.  
 - The two translucent shades (T-Blue and T-Neutral) allow the creation of aesthetic restorations for application in very thin layers over the shades of the VITA® Classical guide, giving the dentist the possibility to reproduce the tooth structure (brighter or darker).  
 - The two translucent shades (T-Blue and T-Neutral) allow the creation of aesthetic restorations for application in very thin layers over the shades of the VITA® Classical guide, giving the dentist the possibility to reproduce the tooth structure (brighter or darker).  
 - The Opallis resin must be light cured layer by layer with a blue light and with a minimum intensity of 400 and 500nm with at least 450 mW/cm2. Do not remove the inhibiting layer as this affects the adhesion of the following layer of resin. The table 1 presents all shades of Opallis resin available, their respective transluence, commercialized amounts and light curing time.  
 - The light curing time described on table 1 must be followed to guarantee the curing depth, conversion degree, shade stability and mechanical properties of the resin. The capacity of the light to go through the resin is associated to translucence. As the Opallis resin presents different degrees of translucence, several studies were accomplished to determine the appropriate light curing time in order to achieve the resin properties and guarantee its quality. In case of insufficient exposure, the resin properties may not achieve the ideal and its last patient.

**Precautions and contraindications**  
 This product is intended to be used only according to this instructions manual;  
 Do not use the product if the patient is allergic to one of the substances present in the composition. In case of allergic reactions, discontinue the use of the product and, if needed, seek medical attention;  
 Avoid the contact of uncured resin with the skin, mucosa and eyes because it may present an irritating effect due to the presence of metacrylates. It is recommended to wear all the Personal Protection Equipment (PPE) while handling the product;  
 Avoid the use of eugenol containing materials, such as liners or temporary restoratives, as they hinder proper light curing of the product;  
 When restoring deep cavities, lining material (such as calcium hydroxide) must be used on the pulp wall.  
 The composite is sensitive to light, so make sure to keep the syringe properly closed when not in use. The resin content should be used right

tightly closed;

- Protect from direct sunlight;
- Store at temperature between 5°C and 30°C / 41°F and 86°F;
- Do not freeze the product.

**Expiration date**
Printed on the product packaging.

**Warnings**
Dental products should be used by authorized professionals only;
Do not use the product after its expiration date;
Follow your country's legislation when disposing the product;
Keep out of the reach of children.

*This material was manufactured for dental use only and must be manipulated according to the manufacturer's instructions of use. The manufacturer is not responsible for any damage caused by misuse or improper manipulation of the product. The user is obligated to prove (before using the product and under his/her responsibility) that this product is compatible with the desired utilization, especially when such utilization is not indicated in the instructions of use. Data description does not constitute any type of warranty and there is not binding.*

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